www.microlife.com
 Email: msa@microlifeusa.com
 Fax: +1 272 442 5377
 Tel: +1 272 442 5353
 1611 Gulf Blvd., 2nd Floor Ste A
 Clearwater, FL 33755 USA
 Microlife USA, Inc.
 North Central / South America
 www.microlife.com
 Email: service@microlife.com
 Fax: 886 2 8797-1283
 Tel: 886 2 8797-1288
 11492 15mw Rd, R.O.C.
 9F, 431 Republic Road, Nanhui
 Microlife Corporation
 Microlife AG
 Europe / Middle-East / Africa
 Distributor
 Email: admin@microlife.ch
 Fax: +41 71 227 70 30
 Tel: +41 71 227 70 30
 ESPortclass 139
 Microlife AG


EN RU UA


Microlife NEB 100

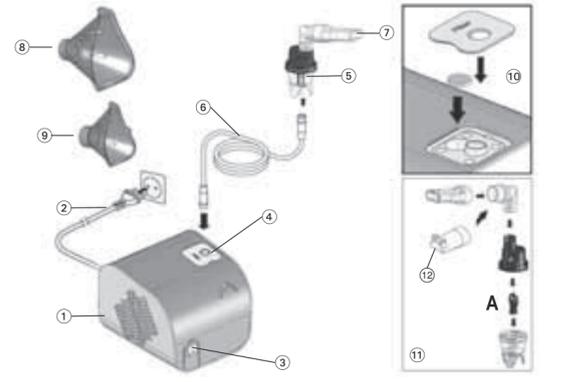NEB 100
 IB NEB 100B RU-V3 1213

Short Instruction

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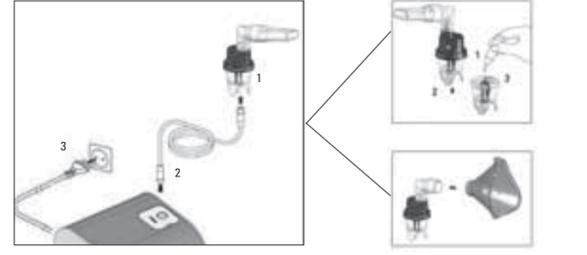CE0434

Compressor Nebuliser

- 1 Piston compressor
- 2 Power lead
- 3 ON/OFF Switch
- 4 Air filter compartment
- 5 Nebuliser
- 6 Air hose
- 7 Mouthpiece
- 8 Adult face mask
- 9 Child face mask
- 10 Replacing air filter
- 11 Assembling nebuliser kit
- 12 Nose piece

Dear Customer,
 This nebuliser is a high-quality device for inhalation therapy for asthma, chronic bronchitis, and other respiratory illnesses.
 The operation and usage of the device is very straightforward. All common liquid medication can be used for inhalation therapy.
 Please read through these instructions carefully so that you understand all functions and safety information. We want you to be happy with your Microlife product. If you have any questions, problems or want to order spare parts please contact Microlife-Customer Service. Your dealer or pharmacy will be able to give you the address of the Microlife dealer in your country. Alternatively, visit the internet at www.microlife.com where you will find a wealth of invaluable information on our products. Retain instructions in a safe place for future reference. Stay healthy – Microlife AG!

Read the instructions carefully before using this device.
 Type B applied part

1. Important Safety Instructions

- This device may only be used for the purposes described in these instructions. The manufacturer cannot be held liable for damage caused by incorrect application.
- This device is not suitable for anaesthesia and lung ventilation.
- This device should only be used with original accessories as shown in these instructions.
- Do not use this device if you think it is damaged or notice anything unusual.
- Never open this device.
- This device comprises sensitive components and must be treated with caution. Observe the storage and operating conditions described in the «Technical Specifications» section.
- Protect it from:
 - water and moisture
 - extreme temperatures
 - impact and dropping
 - contamination and dust
 - direct sunlight
 - heat and cold
- Only use the medication prescribed for you by your doctor and follow your doctor's instructions with regard to dosage, duration and frequency of the therapy.
- Never bend the nebuliser over 60°.
- Ensure that children do not use this device unsupervised; some parts are small enough to be swallowed.
- Electronic devices must be disposed of in accordance with the locally applicable regulations, not with domestic waste.
- Use of this device is not intended as a substitute for a consultation with your doctor.

2. Preparation and Usage of This Device

- Prior to using the device for the first time, we recommend cleaning it as described in the section «Cleaning and Disinfecting».
1. Assemble the nebuliser kit (1). Ensure that all parts are complete.
 2. Fill the nebuliser with the inhalation solution as per your doctor's instructions. Ensure that you do not exceed the maximum level.
 3. Connect the nebuliser with the air hose (6) to the compressor (1) and plug the power lead (2) into the socket (230V~ / 50Hz).
 4. Switch the ON/OFF switch (3) to position «I» to turn on the device and place the mouthpiece (7) in the mouth or fit one of the face masks over mouth and nose.
 - The mouthpiece gives you a better drug delivery to the lungs.
 - Choose between adult (8) or child face mask (9) and make sure that it encloses the mouth and nose area completely.
 - Use all accessories including the nose piece (12) as prescribed by your doctor.
 5. Breathe in and out calmly during the therapy. Sit in a relaxed position with the upper body upright. **Do not lie down while inhaling.** Stop inhalation if you feel unwell.
 6. After completing the inhalation period recommended by your doctor, switch the ON/OFF switch (3) to position «O» to turn off the device and unplug it from the socket.
 7. Empty the remaining medication from the nebuliser and clean the device as described in the section «Cleaning and Disinfecting».

This device was designed for intermittent use of 30 min. On / 30 min. Off. Switch off the device after 30 min. use and wait for another 30 min. before you resume treatment.

3. Cleaning and Disinfecting

- Thoroughly clean all components to remove medication residuals and possible impurities after each treatment. The compressor (1) and the air hose (6) should be cleaned with a clean, moist cloth.
- Always wash your hands well before cleaning and disinfecting the accessories.
 - Do not expose the compressor to water or heat.
 - Replace the air hose for each treatment with a new patient or in case of impurities.
 - Unplug the device before cleaning.

Cleaning with water

- Wash all nebuliser components (except air hose) under warm tap water (max 60 °C) for about 5 minutes adding if necessary a small quantity of detergent following dosage and use limitations as provided by detergent manufacturer.
- Rinse thoroughly making sure that all deposits are washed away and leave to dry.

Disinfecting

- All nebuliser components (except air hose) can be disinfected with chemical disinfectants following dosage and use limitations as provided by disinfectant manufacturer.
- Disinfectants are usually available at pharmacies.

Sterilizing with steam

- All nebuliser components (except air hose and masks) can be heat steam sterilized up to 121 °C (20 min.) or 134 °C (7 min.). EN554/ISO11134.
- The sterilisation packaging must conform to EN868/ISO11607 and be suitable for steam sterilisation.
- After sterilization always let all components cool down to ambient temperature before further use.
- Do not repeat sterilization cycle when components are still warm.

4. Maintenance, Care, and Service

- Order all spare parts from your dealer or pharmacist, or contact Microlife-Service (see foreword).
- We recommend replacing the nebuliser, mouthpiece, and the face masks after 3 months use.
 - Check the filter continually for cleanliness and replace it if dirty or after a maximum of 3 months use. Spare filters are provided with the device.
 - To replace the filter, open the air filter compartment (4) on the top of the device marked «Filter» and exchange the filter (10).
 - Never unscrew the plastic lid.

5. Malfunctions and Actions to take

- The device cannot be switched on
- Ensure the power lead (2) is correctly plugged into the socket.
 - Ensure the ON/OFF switch (3) is in the position «I».
 - The device has automatically switched off due to overheating. Wait until it has cooled down and try again.

EN

- The nebuliser functions poorly or not at all
- Ensure the air hose (6) is correctly connected at both ends.
 - Ensure the air hose is not squashed, bent, dirty or blocked. If necessary, replace with a new one.
 - Ensure the nebuliser (5) is fully assembled and the colored impactor (11)-A is placed correctly.
 - Ensure the required medication has been added.

6. Guarantee

This device is covered by a 3 year guarantee from the date of purchase. The guarantee is valid only on presentation of the guarantee card completed by the dealer (see back) confirming date of purchase or the receipt.

- The guarantee covers only the compressor. The replaceable components like nebuliser, masks, mouthpiece, air hose, and filters are not included.
- Opening or altering the device invalidates the guarantee.
- The guarantee does not cover damage caused by improper handling, accidents or non-compliance with the operating instructions.

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