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IMPORTANT SAFETY INSTRUCTIONS

- The niple thermometer is suitable exclusively for oral measurement of the body temperature of children up to 3-years of age!
- The minimum measurement time until the signalling tone [beep] must be maintained without exception!
- The readings displayed by the niple thermometer may be too low under some conditions (e.g. if the child is crying or persistently breathing through its mouth). Therefore, in case of doubt, or if there are signs of fever (redness of skin, sweating, agitation, unwillingness to drink), please always perform a follow-up rectal measurement.
- This thermometer is not suitable for use as a pacifier!
- Protect the thermometer from hitting and striking!
- Avoid ambient temperatures above 60 °C. NEVER hold the thermometer!
- The thermometer cap/must not be opened! The batteries cannot be changed for safety reasons.
- Use only the commercial disinfectants listed in the section «cleaning and disinfection» to clean the thermometer. The thermometer must be intact when immersed in liquid disinfectant.
- It is not necessary to calibrate the measurement accuracy within the limited 3-years battery life time.

Batteries and electronic instruments must be disposed of in accordance with the locally applicable regulations, not with domestic waste.

TURNING ON THE THERMOMETER

To turn on the thermometer, push the button above the display window; a short signalling tone [beep] signals «thermometer ON.» A display test is performed. All segments should be displayed. Then at an ambient temperature of less than 32 °C, an «L» and a flashing «C» appear at the upper right of the display. The thermometer is now ready for use.

USING THE THERMOMETER

The thermometer is suitable only for oral measurement of the body temperature of children, and it must not be used as a pacifier. Place the thermometer in the child's mouth like a normal baby nipple. Be sure the child keeps the thermometer in his mouth during the measurement, and keep the child's mouth closed as much as possible. The measurement time signalled is normally about 5 minutes. During a measurement, the current temperature is displayed continuously. The «C» symbol flashes during this period of time. If the signalling tone sounds (beep-beep-beep 10 times) and the «C» is no longer flashing, this means that the measured increase in temperature is now less than 0.1 °C in 32 seconds and that the thermometer is ready for reading. **High Temperature Indication:** 10 short beeps and a red LCD backlight alert the patient that he/she may have a temperature above 37.5 °C.

Warning! Certain external factors can lead to measurement inaccuracy in any measurement of body temperature, especially on small children. Therefore, please note the instruction given in the section «Safety instructions». After a measurement, the maximum temperature measured is shown on the display until the thermometer shuts off. To prolong the life of the battery, turn it off after use by pressing the operating button briefly. Otherwise the thermometer will automatically turn off after about 10 minutes.

BATTERY EXHAUSTION

When the «▼» symbol (upside-down triangle) appears in the lower right of the display, the battery is exhausted and thermometer needs to be replaced. For battery disposal please refer to «safety instructions».

	TECHNICAL SPECIFICATIONS
Тип / Тип:	Maximum thermometer
Measurement range / Диапазон измерения:	32.0 °C to 43.9 °C Temp. < 32.0 °C: display «L» for low (too low) Temp. > 43.9 °C: display «H» for high (too high)
Measurement accuracy / Точность измерений:	± 0.1 °C between 34 °C and 42 °C
Operating temperature / Диапазон рабочих температур:	10 °C to 40 °C; 15 - 95 % relative maximum humidity
Self-test / Самоконтроль:	Automatic internal check at a test value of 37.0 °C; if there is a deviation of > 0.1 °C, «ERR» (error) is displayed
Display / Дисплей:	Liquid crystal display (LCD) with 3 digits Smallest unit of display: 0.1 °C
Signalling tone / Звуковой сигнал:	For signalling that the thermometer is ready to use and that the temperature increase is less than 0.1 °C/32 seconds
Memory / Память	For storing the last measured value.
Storage temperature / Температура хранения:	-25 °C to +60 °C; 15 - 95 % relative maximum humidity
Battery / Батарея:	1.5 / 1.55 V; SR 41
References to standards / Ссылка на стандарты:	EN 12470-3, clinical thermometers; ASTM E1112, IEC 60601-1, IEC 60601-1-2 (EMC)

This device complies with the requirements of the Medical Device Directive 93/42/EEC. Technical alterations reserved.

GUARANTEE

We offer a 2-year guarantee after the date of purchase. Any damage caused by improper handling shall not be covered by the guarantee. The battery and packaging are also excluded from the guarantee. All other damage claims excluded. A guarantee claim must be submitted with the purchase receipt. Please pack your defective thermometer well and send with sufficient postage to:

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