MicroLife AG, Espenstrasse 139,
9443 Widenau / Switzerland
Tel. +41 / 71 727 70 30
Fax +41 / 71 727 39
Email admin@microlife.ch
www.microlife.com / www.microlife.ru

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IMPORTANT SAFETY INSTRUCTIONS

- The thermometer is suitable exclusively for measurement of the body temperature!
- The minimum measuring time (the depth) must be maintained!
- The thermometer contains small parts (battery, etc.) that can be swallowed by children. Therefore, the thermometer should be kept out of children's reach.
- Use only the component disintegrates listed in the section «Cleaning and disinfection».
- Avoid ambient temperatures above 60 °C. NEVER boil the thermometer!
- Use only the thermometer recommended by the manufacturer in liquid distribution.
- Do not immerse the thermometer in water when immersed in liquid distribution.
- The manufacturer recommends verifying the accuracy by an authorized laboratory every 2 years.
- Batteries and electronic instruments must be disposed of in accordance with the locally applicable regulations, not with domestic waste.

TURNING ON THE THERMOMETER

To turn on the thermometer, push the ON/OFF button on the top; a short beep signals «thermometer ON». A display will appear at the right of the display. After 3 more seconds, this value will disappear and the thermometer will start its measurement.

USING THE THERMOMETER

Choose the preferred measuring method. During a measurement, the current temperature is displayed continuously. The «C» symbol flashes during this period of time. If the signal tone sounds (beep-beep 10 times) and the temperature is ready for reading, 10 seconds after the last measurement, the thermometer will automatically display the stored value. After 3 more seconds, this value will disappear and the thermometer will start a new measurement.

ILLUMINATED DISPLAY / NIGHT GLOW FEATURE

If you hold the ON/OFF button for more than 3 seconds, when turning the thermometer on, the maximum temperature stored automatically will be displayed. At the same time, a small «M» for memory will appear at the right of the display. After 3 more seconds, this value will disappear and the thermometer will start a new measurement.

STORAGE OF MEASURED VALUES

The thermometer will automatically turn off after about 10 minutes. The thermometer will automatically display the stored value. After 3 more seconds, this value will disappear and the thermometer will start a new measurement.

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